

**THE CHAS
CATH-MATE™**


Catheter Holder - *Latex Free*

U.S. Patents #5,352,209 #6,645,185B2

Reorder Number

0814-8200

PROPER APPLICATION ILLUSTRATIONS

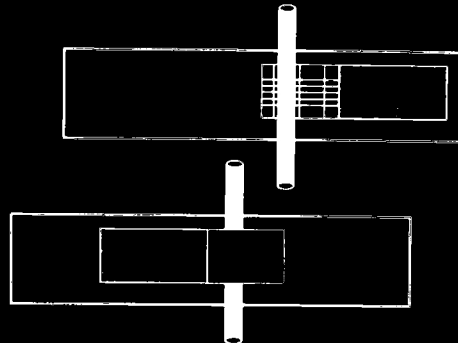

1. Position and secure leg strap to patients thigh
2. Align and position catheter on patented grip securing tube strap.
3. Slightly stretch elastic and secure strap across the catheter and fasten to leg strap. Catheter is now firmly secured and should not slip

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products
1200 Trapp Road • Eagan, Minnesota USA 55121
651/683-1111 • Fax 651/688-9855
800/328-1095 • Fax 800/279-7934