

Fritz Sling & Swathe Universal

Application Instructions

1. Position sling against rib cage. Bring web straps straight up and over the patient's shoulders. Place affected arm through the sling (figure 1).



figure 1

2. Criss cross both straps against the patient's back (figure 2).

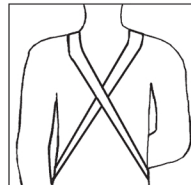


figure 2

3. Attach hook straps to the front of the sling in a criss cross motion supporting the affected arm (figure 3). Straps may be trimmed if necessary (alligator hook attachment piece is removable at the end of both straps and can be re-attached after trimming).



figure 3

4. Re open pouch. Secure arm against the body by attaching the swathe strap to the sling using the inside foam pad. Place one end of the strap to the sling. Encircle the affected arm and bring around the patient's back and under the non-affected arm (figure 4). Secure pouch in closed position (figure 5).

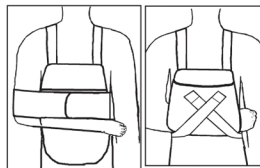


figure 4

figure 5

Washing Instructions

Join hook and loop. Machine wash in cold water with mild detergent. Air dry.

Reorder No.	Size	Swathe Length
0814 0168	Universal	Up to 54" 137.16cm

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products
1200 Trapp Road • Eagan, Minnesota USA 55121
651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934
e-mail sales@birdcronin.com • www.birdcronin.com

©2010 Bird & Cronin, Inc. All Rights Reserved - 10000168