


ANKLIZER® Low Profile Support

Application Instructions


Reorder No.	Size
0814 0391	X-Small
0814 0392	Small
0814 0394	Medium
0814 0395	Large
0814 0396	X-Large




1. Place foot and lower leg inside the softgood. Close firmly by applying the hook fasteners to the outside of the softgood.

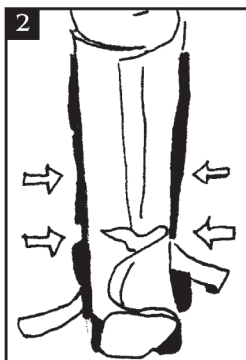

2. Insert foot and lower leg into brace. Ankle should be carefully centered between the uprights to ensure comfort and proper support.

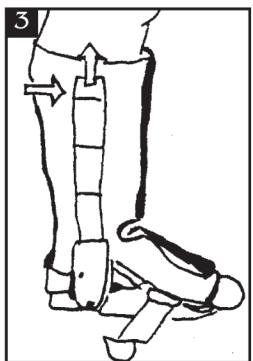

3. After verifying that the uprights are centered over the malleoli, remove the plastic sleeves covering the uprights to lock the softgood into place.

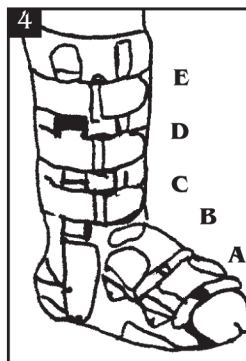

4. Secure the hook and loop straps. Begin with strap "A", then "B" ending with leg strap "E". Straps should firmly secure the foot, ankle and lower leg in position. Do not over tighten these straps.

Properly fitted, the **Anklizer™** will provide maximum support and total patient comfort. To eliminate rubbing or loose areas, use the included foam pads as necessary. Pads can be inserted inside the softgood or between the softgood and the base or uprights.

To remove leg from the **Anklizer™**, release the hook and loop straps and then open the softgood. Gently pull the leg and foot forward and out of the softgood. The softgood should remain in position in the brace, unless removed for washing.

CAUTION

- This brace should not be removed without specific instruction from a physician
- Federal law restricts the device to sale by or on the order of a physician or duly licensed practitioner

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com • www.birdcronin.com