


BIRD & CRONIN INC.

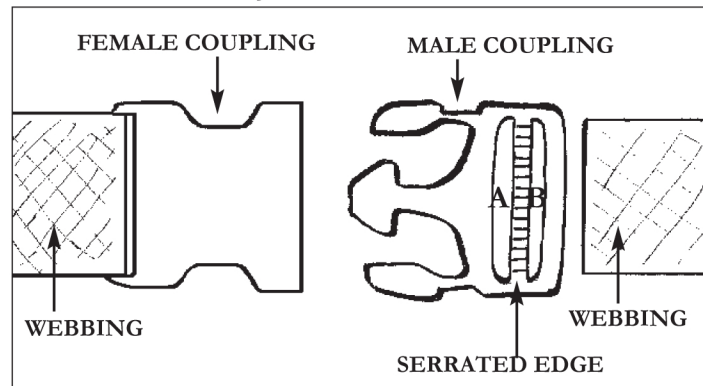

**TO PREVENT BELT SLIPPAGE,
PLEASE FOLLOW DIRECTIONS CAREFULLY**

1. Webbing must be inserted under male component of buckle, serrated edge up, through slot "A", loop over serrated edge and down through slot "B".
2. Adjust belt by pulling strap through loop "B". Once adjusted, hold both loose strap and belt strap together for added security.
3. Buckle should be positioned in the middle of patients waist to eliminate possibility of webbing pulling perpendicular to the buckle.

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

800/328-1095 • Fax 800/279-7934 • 651/683-1111 • Fax 651/688-9855

10006452