

## B-Cool® Arthroscopic Knee Wrap - Universal

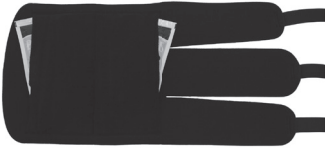


Figure 1



Figure 2



Figure 3



Figure 4

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

**Intended for single patient use.**



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Unfasten the leg straps and unfold the knee wrap.
2. Insert one B-Cool® ice pack into each pocket. (Figure 1)
3. Sit on a flat surface with the affected leg in full extension.
4. Place knee wrap on top of the leg and center the opening over the patella (knee cap). (Figure 2)
5. Wrap and secure the middle strap first. (Figure 3)
6. Wrap and secure remaining straps. (Figure 4)
7. Adjust as needed.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**

