

## Striped Walking Belt

**NOTE:** Instructions cover multiple product families, images may vary.

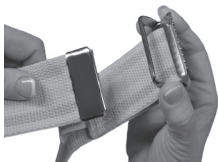


Figure 1

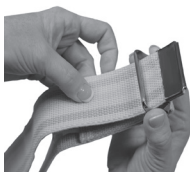


Figure 2

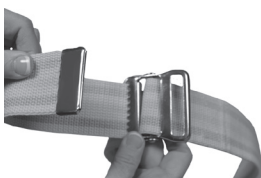


Figure 3

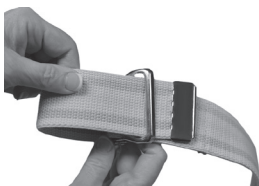


Figure 4

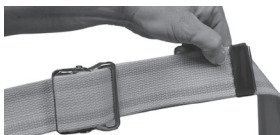


Figure 5

1. Wrap the belt around the patient's mid-section, with buckle facing outward.
2. Feed the metal tipped end of the belt through the buckle and over the teeth (Figure 1 & 2).
3. Pull the tip until snug so the teeth can secure it (Figure 3).
4. Bring tip of belt across the front of the buckle and thread through the non-teeth portion of the buckle (Figure 4).
5. Adjust the belt until it is snug, but not uncomfortable for the patient.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**



Bird & Cronin LLC  
1200 Trapp Road · Eagan, MN USA 55121  
800-328-1095 · [www.birdcronin.com](http://www.birdcronin.com)