

Wrist-Mate™ Wrist Brace



Figure 1



Figure 2



Figure 3



Figure 4



Figure 5

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Unfasten straps and separate brace into two pieces. (Figure 1)
2. Position affected hand into component A. The curved stay should be positioned at the crease of the palm. (Figure 2)
3. Fold portion of component A over wrist and forearm.
4. Place component B on dorsal (top) side of hand. Attach component B to component A using hook and loop. (Figure 3)
5. Secure bottom strap(s) to top of component B. (Figure 4)
6. Thread mid-strap through link securing until desired level of fit and compression is reached.
7. Bring one strap around pinky finger and attach to the top of brace (component B). Place second strap between thumb and index finger, forming an “X” and secure. (Figure 5)

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at 800-328-1095.

NOTE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

