


Prody™ Drainage Bulb Holder

Designed to securely hold suction reservoirs during post-operative recovery.

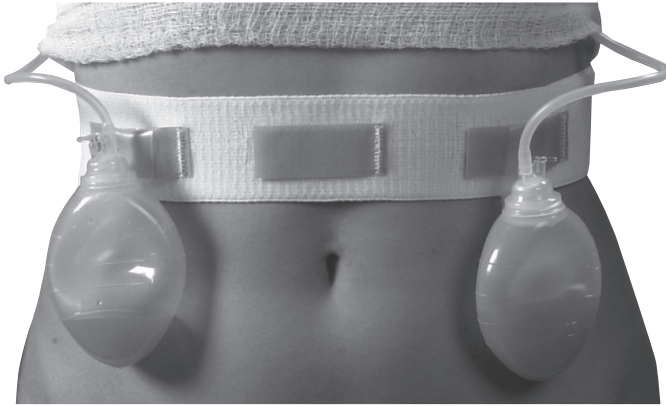

U.S. Patent #6,610,032B1

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Stretch Prody Drainage Bulb Holder around body and press white hook tab to secure.
2. Rotate Prody Drainage Bulb Holder until the locking tabs are lined up with drainage bulbs.
3. Lift blue hook locking tab from holder and thread through loop attached to drainage bulb. Press hook to secure. Repeat as necessary for up to four drainage bulbs.

CARE INSTRUCTIONS

Secure Hook to holder to close. Hand wash with cold water. Air Dry.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH
natural rubber latex**

