

LUMBO ROLL

NOTE: Instructions cover multiple product families, images may vary.

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.

APPLICATION INSTRUCTIONS

1. Sit in the seat as far back as possible.
2. Lean forward and place the device horizontally across the lower back, just above the belt line, so that the hollow of the back is firmly supported when you lean back.
3. If necessary, adjust position by moving the device up or down in no more than one inch increments at a time.

CARE INSTRUCTIONS

1. Fasten all hook-and-loop closures.
2. Hand wash with mild detergent in cold water. Do not use fabric softener.
3. Allow to air dry. Do not expose to direct heat sources such as dyers, radiator, or stove.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH
natural rubber latex**

