

ENDO-MATE™

Endotracheal Tube Holder

Application Instructions



Reorder Number
0814-8115

1. Unfold tube holder. Position rubber gripping area of tube holder strap face up under the endotracheal tube of the patient.
2. Bring shorter strap up and over the tube to the opposite side.
3. Bring the longer strap up and over the tube and over the shorter strap. Fasten snugly and securely to the white hook closure.
4. Bring the longer strap around and under the patient's neck and secure comfortably to the long white hook tab on the shorter strap.
5. Excess strap may be trimmed if desired.

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products
1200 Trapp Road • Eagan, Minnesota USA 55121
800/328-1095 • Fax 800/278-7934 • 651/683-1111 • Fax 651/688-9855
e-mail sales@birdcronin.com • www.birdcronin.com

©2007 Bird & Cronin, Inc. All Rights Reserved - 10008115