

Super Sling 2.0 - Immobilization and Support of the Shoulder



APPLICATION INSTRUCTIONS

Please read these instructions completely and carefully before applying this product.
Correct application is important for proper function of this brace.



Latex Free

Sling and Pillow


A. Wrap and secure waist belt with sling around the patient's waist. Open wrist and forearm strap clips and elbow closure on the sling. Trim excess strap to desired length (Image A-1).

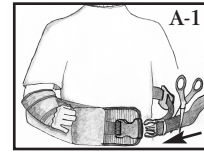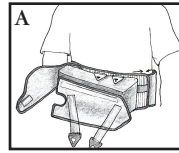

B. Place forearm into sling so that the hand is supported by the sling. Fasten wrist strap clip and then the forearm strap clip. The wrist and forearm should be held snugly against the pillow. Hook/loop straps may be adjusted as appropriate (Image B-1). Secure the posterior elbow strap to the waist strap to hold the elbow against the pillow.


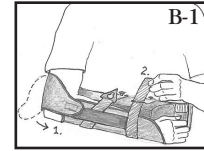
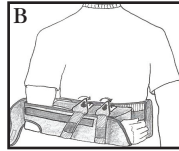


C. A properly applied Super Sling 2.0.

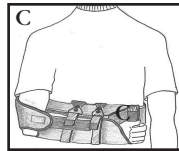

Shoulder Strap (optional)



D. Attach shoulder strap to the front of the abduction pillow, with strap oriented over unaffected shoulder and attach strap to the waist belt under the unaffected shoulder (Image D-1). Adjust strap as necessary for a proper fit.

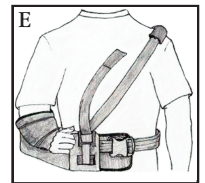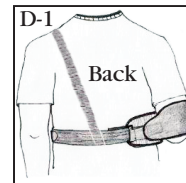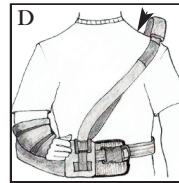

E. Excess strap material may be trimmed by removing y-tab, trim the strap length then reapply y-tab.



Conversion to a Shoulder Immobilizer

F. Detach sling and sling straps from pillow. Detach pillow from sling and remove strap clips from sling straps.

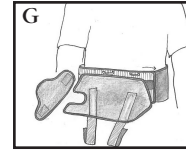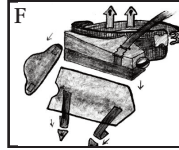


G. Attach sling to waist just below the plastic D-rings.

H. Wrap and secure waist belt with sling around the patient. Place forearm into sling so that the hand is supported by the sling. Feed wrist strap through D-ring and secure. Feed forearm strap through D-ring and secure.

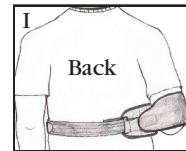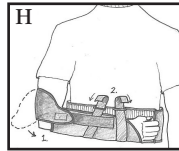

I. Secure the posterior elbow strap to the waist strap to hold the elbow against the body.

J. A properly applied Super Sling 2.0 without the pillow.

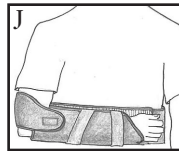

Use and Care: Hand wash in cold water using mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

Warning: The product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes or any unusual reactions while using this product, consult your medical professional immediately.

Caution: Federal Law (U.S.A) restricts this device to sale by or on the order of licensed health care professional. For single patient use only.

Reorder No.	Description	Size	Waist Size
0814 8241	Left	Universal	up to 64"
0814 8242	Right	Universal	up to 64"

Manufactured For

BIRD & CRONIN INC.

Quality Medical Products To Help Improve Your life
1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com

©2013 Bird & Cronin, Inc. All Rights Reserved - 10008241A